

N95 Respirator

Product Specifications



Product Description

4-layer mask with ear loops. Melt blown cloth and non-woven material. White.

Basic Specifications

- Item Number: HMA215WHJ2H
- Material: Melt Blown Cloth and Non-Woven Material
- Color: White
- Pack Size: 20

Additional Features

- Disposable
- Non-Sterile
- Length of head straps: 10"
- Size: Universal



Quality Standard

- CE Compliant under Medical Devices Directive 93/42/EEC
- NIOSH Approved
- ASTM Level 1
- BFE \geq 95%
- PFE \geq 95%

Instructions

Position the respirator in your hands with the nose piece at your fingertips. Cup the respirator in your hand allowing the headbands to hang below your hand. Hold the respirator under your chin with the nosepiece up. The top strap goes over and rests at the top back of your head. The bottom strap is positioned around the neck and below the ears. Do not crisscross straps. Place your fingertips from both hands at the top of the metal bridge. Slide fingertips down both sides of the metal strip to mold the nose area to the shape of your nose. To remove: DO NOT TOUCH the front of the respirator! It may be contaminated! Remove by pulling the bottom strap over back of head, followed by the top strap, without touching the respirator. Discard after use.

To place an order email healthorders@foh.cc or call 305-757-7940 ext.3 for Customer Service.
For inquiries email health@foh.cc www.fohhealth.com